


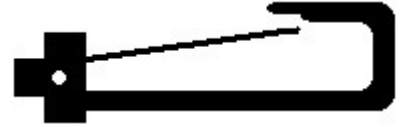

ATTACHING THE WEBBING TO THE SEAT BACK

(See Opposite Side For Diagram)

- 1 Remove the webbing from the pouch, undo the clips, and lay out on a table so that the clip is face-up on the webbing.
- 2 Position yourself so you are facing the back of the seat that will hold the cylinder.
- 3 Lay the webbing over the seat so that the clip is facing out and down from the top of the back seat, facing you.
- 4 Drape the long part of the webbing along with the cross piece, over the front of the seat. If the seat has a headrest, raise the headrest and run the webbing underneath. The cross piece should be high across the front of the seatback.
- 5 The remaining part of the webbing strap below the clip should hang down the back of the seatback.
- 6 The vertical webbing on the seat front has two buckles—a large buckle and a smaller one. Push this strap between the seat cushion and the front of the seat-back completely.
- 7 Grasp the larger buckle [from Step 6] and attach it to the buckle from the webbing strap hanging down the seat back from Step 5.
- 8 Tighten this assembly. The vertical position of the webbing is now securely attached to the seat.
- 9 Find the triangular fitting on the carrier pouch and attach it to the clip as in Step 3. The carrying handle of the pouch should be facing you, toward the back of the airplane.
- 10 Bring the two horizontal straps from the front of the seat and attach them to the buckles on the pouch.



- 11 The remaining buckle hanging down from the seat back is inserted between the webbing and the bottom of the pouch, and attached to the smaller buckle on the bottom of the pouch.
- 12 Install the cylinder in the pouch.
- 13 Tighten all webbing straps to secure the pouch.

