



SHAW-AEROX  
LLC



25190 Bernwood Dr.  
Bonita Springs, FL.  
34135

800 237 6902  
Fax 207 637 2329  
aerox@aerox.com  
www.aerox.com

**aerox**

## ATTACHING THE WEBBING TO THE SEAT BACK

(See Opposite Side For Diagram)

- 1 Position yourself so you are facing the back of the seat that will hold the cylinder.
- 2 Remove the webbing from the pouch.
- 3 Find the webbing strap with the clip.
- 4 Hang the clip down and facing you.
- 5 Drape the webbing with the cross piece [longer strap] over the front of the seat. The webbing strap below the clip hangs down the seat back.
- 6 The vertical webbing on the seat front has two straps, each with a buckle. Push this strap between the seat cushion and the seat back completely.
- 7 Grasp the larger buckle [from Step 5] and attach it to the buckle from the webbing strap hanging down the seat back from Step 4.
- 8 Tighten this assembly. The vertical position of the webbing is now securely attached to the seat.
- 9 Find the triangular fitting on the carrier pouch and attach it to the clip as in Step 3.
- 10 Bring the two horizontal straps from the front of the seat and attach them to the buckles on the pouch.
- 11 The remaining buckle hanging down from the seat back is inserted between the webbing and the bottom of the pouch, and attached to the buckle on the pouch.
- 12 Install the cylinder in the pouch.
- 13 Tighten all webbing straps to secure the pouch.



